

Nonwoven MOUTH AND NOSE MASK


Reusable & washable up to 5 times



- OEKO-TEX-certified non-woven fabric ensures optimum air circulation
- Synthetic two-layer non-woven (2x50g)
- Elastic band with very good rubber elasticity
- Hypoallergenic materials
- Washable at max 70° up to 5 times
- Ironable at max 150°
- Industrial UV and ozone disinfection
- Completely manufactured in the European Union
- **For private use only**
- **Non-medical use**
- Art. Nr. DI 2020M101 : availability
 - in carton of 50 pcs*
 - individually wrapped in carton of 250 pcs*(* stock may vary)



Handelsstrasse, 16 – 4700 EUPEN
Tel. 087/59.51.70 – Fax 087/59.51.79 –
info@filterservice.be - orders@filterservice.be

	PRODUCT SPECIFICATION	Doc. N°DS09812 Rev. Of April 2020
	Mask Art.Nr 2020M101	Granted by Quality Assurance
PRODUCT:	MOUTH & NOSE MASK reusable & washable up to 5 times	

DESCRIPTION:

Mouth & Nose Mask for Private use only.
Non-medical use.

GENERAL PROPERTIES:

OEKO-TEX certified Standard 100 nonwoven fabric, ensures optimum air circulation
One size fits all, adapted to the morphology of an adult face.
Light, soft and comfortable, it is easily accepted and worn by the user.
Offers an excellent adaptation to the face and therefore a good level of protection.
Without Latex, Silicone or PVC, it is made of hypoallergenic materials and offers a very good Abrasion resistance.
Completely produced in the EU.
FSC certified

TECHNICAL PROPERTIES:

Industrial UV and Ozone disinfection treatment
ISO 9237 Permeability of fabrics to air: 4000 l/m² each layer,
Sewn from 2 layers, grammage per layer m²: 50g
Ironable on temperature maximum 150 degrees Washable: see wash recommendations below

COMPOSITION:

Synthetic two-layers (2x50g) nonwoven fabric made of mix from polyester, polypropylene, polyamide, viscose and resin.
Elastic band made of rubber, with good elasticity. Color of the elastic may change.

USAGE, WASHING & DISINFECTION RECOMMENDATIONS

Usage

The mask should be washed whenever it is soiled, wet or improperly positioned on the face.
It should not be placed in a waiting position on the forehead or under the chin during and after use.
Wearing time may not exceed 4 hours in a single day.

To put on the mask

- Before handling the mask, wash hands with soap and water or with a hydroalcoholic solution.
- Hold the mask from the outside and pass the elastics at the back of the head, on either side of the ears, without crossing them.
- Place the mask on the face with the horizontal seam on the nose.
- Lowering the bottom of the mask under the chin
- Once adjusted, do not touch the mask with your hands.
- Whenever the mask is touched, the user should wash hands with soap and water or with a hydroalcoholic solution

To remove the mask

- Before handling the mask, wash hands with soap and water or with a hydroalcoholic solution.
- Remove the mask by grasping the elastic bands of the strap set from behind without touching the front part of the mask.

To wash

- Use an usual detergent or liquid soap, preferably perform a gentle hand wash.
- The mask shall be allowed to dry completely before ironing.
- It is then recommended to spray an alcoholic solution on it to disinfect effectively.



SHELF LIFE & STORAGE

The shelf life of the mask is 2 years from its production date.
Recommended storage conditions (storage in a cool and dry place, temperature 8-25°C and relative humidity <70%, protected from direct sunlight)

The fabric is produced under different certifications such as IATF, ISO 9001 -2015 and is FSC certified.

Date: April 2020 Version nr.01